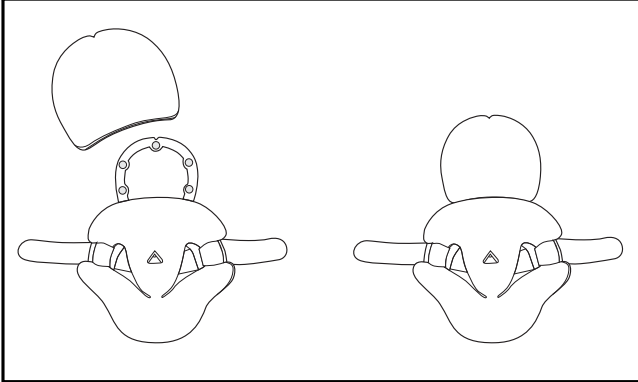
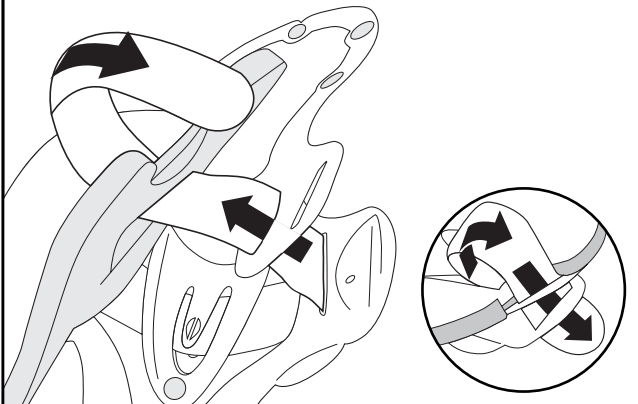
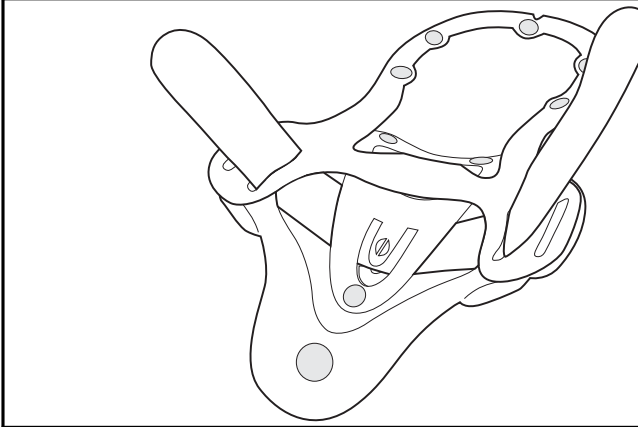
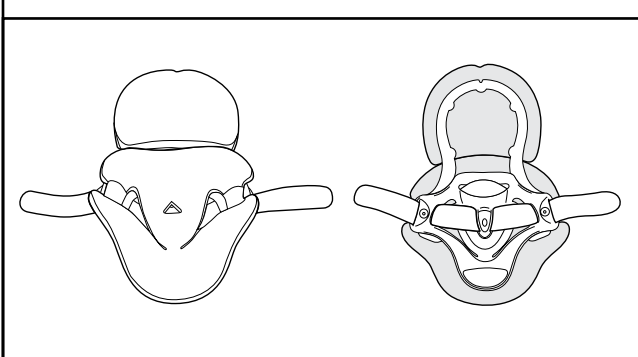
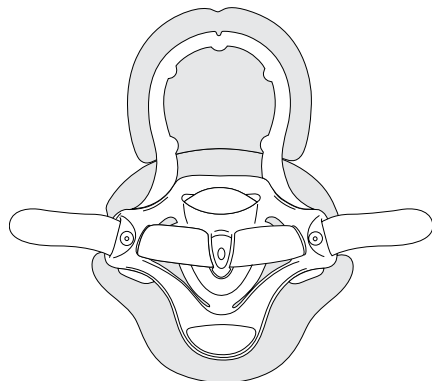
VISTA® ICU BACK PANEL

- VISTA® ICU HINTERE PLATTE
- PANEL POSTERIOR VISTA® ICU
- PANNELLO POSTERIORE VISTA® ICU
- PANNEAU ARRIÈRE VISTA® ICU
- VISTA® ICU RUGPANEEL
- ZADNÍ PANEL SYSTÉMU VISTA® ICU
- VISTA® ICU BAGPANEL
- ΠΙΣΩ ΦΥΛΛΟ VISTA® ICU
- VISTA® ICU BACK PANEL
- PANEL TYLNY VISTA® ICU
- VISTA® ICU TAKAPANEELI
- VISTA® ICU ВАКPANEL
- ЗАДНЯЯ ПАНЕЛЬ VISTA® ICU
- PAINEL POSTERIOR VISTA® ICU
- VISTA® ICU 背板

ENGLISH

INTENDED USE
For use with the Vista Cervical Collar on patients who will remain supine for extended periods of time in clinical settings.

INDICATIONS
Motion restriction due to an injury or instability on supine patients for an extended period of time.

CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS
Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

MATERIALS
Nylon, Cotton, Polyurethane, HDPE.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
Product may be safely disposed in accordance with local laws.

REPORTING
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location. A listing of authorized distributors in your area is available upon request.

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©2023 Aspen Medical Products, LLC. All Rights Reserved.
Aspen Medical Products, LLC heeft geen enkele verplichting onder deze beperkte garantie in het geval:
(a) Het product niet gekocht is bij Aspen Medical Products, LLC of via de geautoriseerde distributiekanalen;
(c) Het product in het product worden geplaatst die niet door Aspen Medical Products, LLC worden geleverd; of
(d) Het product niet wordt gebruikt in overeenstemming met de gebruiksvorschriften van Aspen Medical Products, LLC.