
## JURACMC - Jura CMC Thumb Brace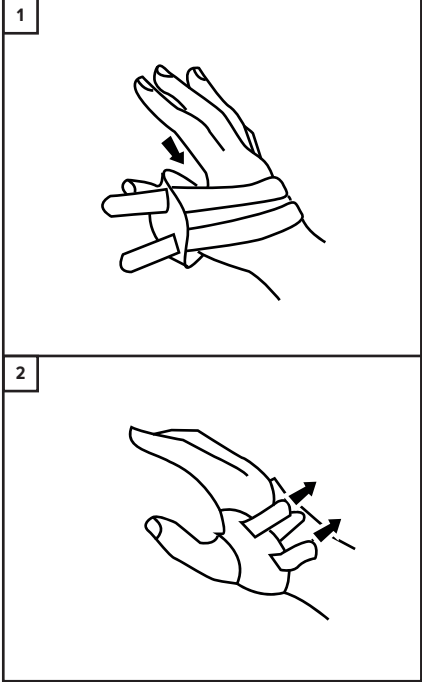

#### ENGLISH

##### Product Description

A thumb orthosis which provides adjustable support to the thumb carpometacarpal (CMC) joint. Inner mouldable aluminium stay offers extra support and stability of the base of the thumb.

##### Usage Information (See diagram)

- Open hook/loop fastening on the back of the orthosis.
- Position thumb in brace. Close hook/loop fastening. Take care not to over tighten straps, re-adjust for comfort as required while ensuring orthosis is in correct position.

Note: the inner mouldable stay can be shaped and adapted for

stabilisation of the base of the thumb.

##### Sizes and Fit

Size	Wrist Circumference (cm)
S	16 – 19 cm
M	19 – 23 cm
L	23 – 26 cm

##### Indications for Use

- Predominately designed to provide support for thumb CMC osteoarthritis.
- May also provide thumb CMC joint support for instability and soft tissue injuries / conditions.

It is recommended that individuals seek guidance from a medial practitioner regarding the most appropriate orthosis.

##### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

##### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

##### Product Composition

TPU, Stainless Steel, Nylon.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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