
## JRWBL - Jura Black Wrist Brace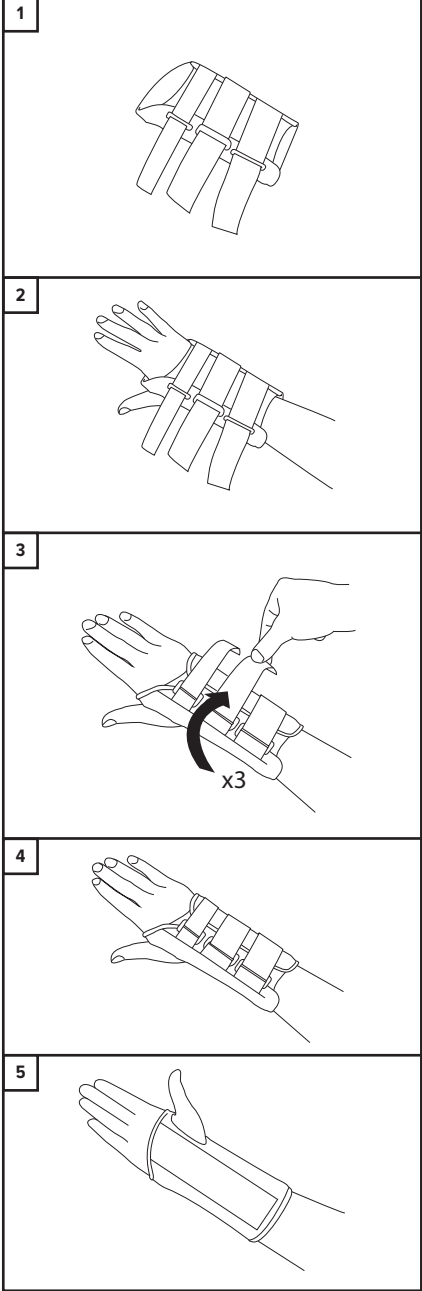

#### ENGLISH

##### Product Description

A wrist orthosis for immobilisation / support. Adjustable stay to help immobilise the wrist. Easy 'slip-on' design with D-ring closure straps.

##### Usage Information (See diagram)

- Open hook/loop fastening and open brace out flat.
- Position hand in brace.
- Close the hook/loop D-ring fastenings around the wrist.
- Take care not to over tighten straps, re-adjust for comfort as required while ensuring brace is in correct position.
- Brace fitting complete.

##### Sizes and Fit

Size	Wrist Circumference
XS	Up to 15 cm
S	15 – 17 cm
M	17 – 19 cm
L	19 – 22 cm
XL	23 – 25 cm

##### Indications for Use

- Provides wrist immobilisation / support for an array of injuries, conditions, pre- and post-operations such as:
  - Stable fractures.
  - Sprains and strains.
  - Soft tissue injuries and conditions.
  - Arthritis.
  - Peripheral nerve pathologies.
  - Slide-on design with D-ring closures allow easy application especially for patients with reduced dexterity.

##### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

##### Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Close over all Hook/Loop straps.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check Hook/Loop fasteners are operating securely.

##### Product Composition

Polyester, Polyurethane, Nylon, Spandex, Polypropylene, POM, Aluminium.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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