# Aspen<sup>®</sup> MEDICAL PRODUCTS

## ASPEN<sup>®</sup> PEDIATRIC COLLAR

PEDIATRIC CERVICAL COLLAR  
PÄDIATRISCHE ZERVIKALORTHESE  
CUELLO CERVICAL PEDIÁTRICO  
COLLARE CERVICALE PEDIATRICO  
COLLIER CERVICALE PEDIATRICE  
PEDIATRISCHE NEKBRACE  
DĚTSKÝ KRČNÍ LÍMEC  
PÆDIATRISK HALSHVIRVELKRAVE  
ΠΑΙΔΙΑΤΡΙΚΟ ΑΥΧΗΝΙΚΟ ΚΟΛΛΑΡΟ  
PEDIATRISK CERVICAL KRAGE  
KOLNIERZ ORTOPEDYCZNY PEDIATRYCZNY  
LASTEN KAULUTUKI  
PEDIATRISK NACKKRAGE  
ШЕИЊИЙ КОРЦЕТ ДЛЯ ДЕТЕЙ  
COLLAR CERVICAL PEDIÁTRICO  
儿科颈托

ASPEN PD1

ASPEN PD2

ASPEN PD3

ASPEN PD4

ASPEN PD5

ASPEN ADULT SHORT

MD MEDICAL DEVICE

SINGLE PATIENT - MULTIPLE USE

CAUTION


EMERGO EUROPE  
Westervoortseidijk 60,  
6827 AT Arnhem  
The Netherlands

Pat. aspemp.com/patents


Aspen Medical Products, LLC  
6481 Oak Canyon, Irvine, CA 92618-5202 USA  
aspemp.com • +1-800-295-2776 • +1-949-681-0200

©2023 Aspen Medical Products, LLC. All Rights Reserved.  
RC0075K revised 03/2023


1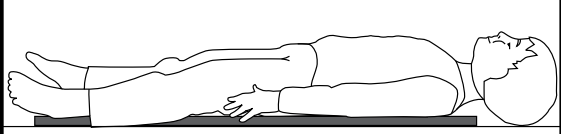


2A

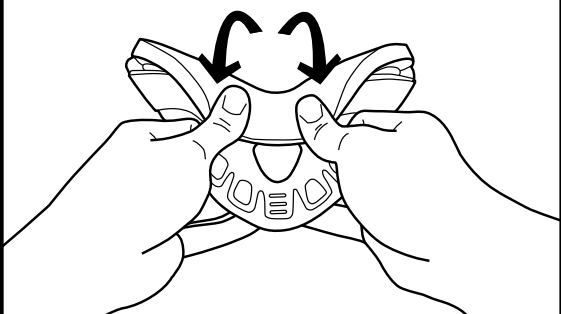
2B

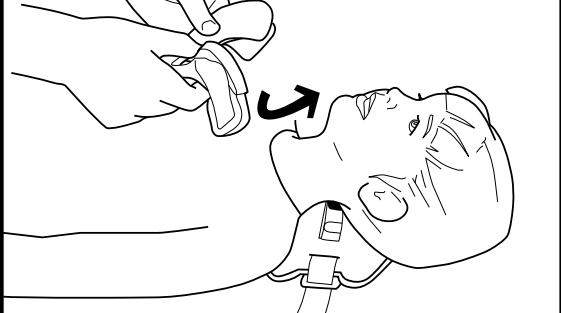
3A

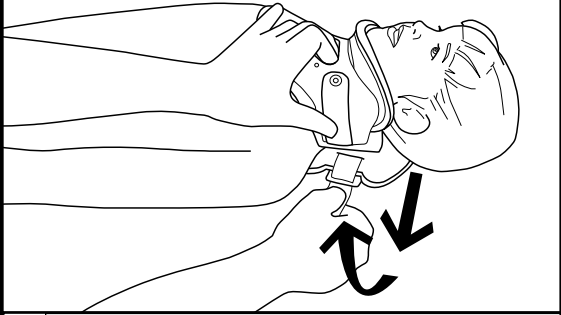3B

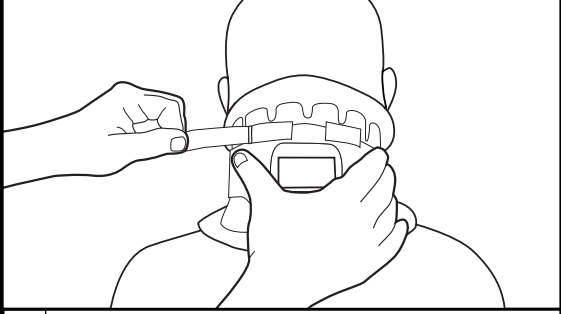4

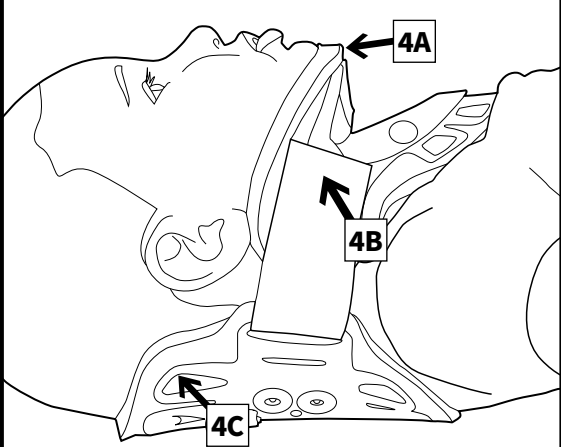ENGLISH

1. SIZING

A child's height, weight and age influence the proper cervical collar sizing. Refer to the sizing chart to estimate the proper size.

SIZING GUIDE				
Size	Age	Weight	Stature	
PD1	1 – 18 months	11 – 29 lbs 5 – 13 kg	21 – 33 in 53 – 84 cm	
PD2	9 – 24 months	22 – 33 lbs 10 – 15 kg	29 – 37 in 74 – 94 cm	
PD3	1 – 3 years	24 – 36 lbs 11 – 16 kg	33 – 40 in 84 – 102 cm	
PD4	2 – 5 years	26 – 42 lbs 12 – 19 kg	35 – 45 in 89 – 114 cm	
PD5	3 – 6 years	27 – 54 lbs 12 – 25 kg	37 – 48 in 94 – 122 cm	
Adult Short	5 years & up	36 lbs & up 16 kg & up	42 in 107 cm & up	

Estimate the proper size using the information above. Remember, too tall a collar can cause hyperextension, while too short a collar can affect the airway and provides less motion restriction.

2. POSITIONING

A. Position the panel behind the neck.  
B. Squeeze the collar front in half horizontally.  
C. Scoop the front panel up and under the chin so the chin piece comes to the front edge of the chin. Bring the sides up over the shoulders and loosely place the back panel loop on the front panel to hold it in place.

3. SECURE

A. While holding the trach hole with one hand, peel the back panel loop outwards until all slack is out. Next, secure the back panel loop onto the side of the front panel. Repeat on the opposite side, and alternate sides until the collar is secure and centered.  
B. (Ambulatory only) The PD 3, PD 4 and PD 5 have an occipital support strap on the back panel. Adjust by squeezing the back panel to the neck and pulling the straps until all slack is out.

4. CHECK THE FIT

A. CHIN DOTS HIDDEN? The chin piece plastic should be flush with the end of the chin, hiding the "chin-dots" indicator on the chin pad.  
B. STRAP PAST MARK? Remove all slack. Evenly tighten and re-tighten the straps, ensuring the loop strap extends past the green mark for a secure fit and front/back panel overlap.  
C. FLEX TABS BENT? If the collar has been properly sized and tightened, the Flex Tabs will be "loaded" to spread the support. (Note: The inner trach bar should never touch the airway.)

INTENDED USE

To provide motion restriction of the cervical spine.

INDICATIONS

Decompressive procedures, Fracture management, Postsurgical stabilization, Whiplash, Sprains/strains, Cervicogenic headache, Cervical disc syndrome, Musculoskeletal pain, Radiculopathy.

CONTRAINDICATIONS

Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS

Hand wash only at 30° C. Wash with mild detergent; Air dry. Do not use bleach or other chemicals.

MATERIALS

HDPE, Nylon, Polyurethane, Cotton.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL

Product may be safely disposed in accordance with local laws.

REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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